


Breathable fluid-proof inner cover made with multi-directional stretch fabric, for immersion, wipes clean between washes

- (A) **Radius cut rear** for sacral support
- (B) **Pre-ischial support** prevents sliding while providing pelvic support
- (C) **Air mesh cover** for comfort & low temperatures
- (D) **Moderate abductor/adductor contour** for mid-line positioning of the femurs
- (E) **Beveled front edge** for foot propellers or users who want to sit with a tight front frame angle or who have tight ham strings
- (E) **Textured non-skid bottom** with loop Velcro & carry strap



Anatomically shaped SPS (Selectively Perforated Softening) perforations are anatomically positioned to support the ischial/sacral area to prevent sliding and shear and maintain pelvic positioning. Provides immersion and lower pressures to the bony prominences while supporting surrounding areas that can take a higher load.

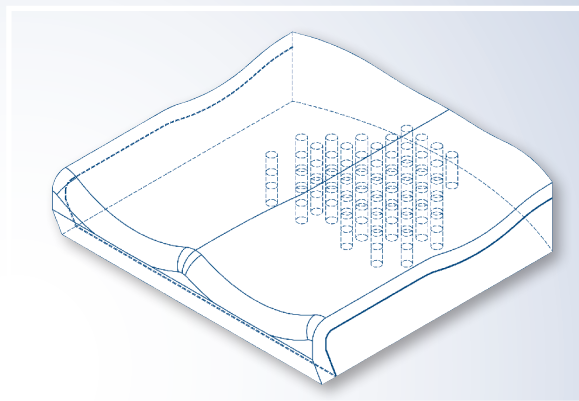

CUSHIONS

nxt™ cushions have been designed to ensure that pre-ischial support prevents sliding and provides pelvic support. The moderate abductor/adductor contour ensures mid-line positioning of the femurs.

The radius at the rear provides sacral support in addition to matching the contour of modular back supports. The bevel on the front edge accommodates foot propellers, users who wish to sit with a tight front frame angle or those with tight hamstrings.

nxt cushions come standard with an outer and inner cover. The outer cover is an air mesh cover provides comfort and keeps temperatures low. It has a textured non-skid bottom with loop Velcro and a carry strap.

The inner cover is fluid proof yet breathable. It is made of multi-directional stretch fabric allowing proper immersion. It has a zipper to allow for easy removal of the foam and can be wiped clean after each use. A fluid proof incontinence cover is an available option.

PELVIC OBLIQUITY/LATERAL PADS

Available in 0.5" (1.3cm) or can be doubled to 1" (2.5cm), are easy to install in a variety of locations with double-sided tape and trimmed to fit. Can be used as a pelvic obliquity build up or adduction for thighs.



SOLID SEAT INSERT

Slim profile solid seat inserts are available to provide a firm sitting surface and are contoured to eliminate the hammock effect when cushions are used on a sling seat.



nxt™ BioFit™ CUSHION

The *nxt BioFit* multi-layer design provides deep immersion and support for skin protection for users at medium to high risk of skin break down, by redistributing pressure over the full sitting surface of the cushion.

WHAT'S INCLUDED

- Foam cushion
- Inner cover
- Outer cover

ACCESSORIES & OPTIONS

- Fluid proof incontinent cover
- Replacement air mesh outer cover
- Replacement inner cover
- Pelvic obliquity/lateral pad
- Solid seat insert

SPECIFICATIONS

Cushion Weight*	2.4lbs (1.09kg)
Depth	10 - 22" (25.4 - 55.9cm)
Width	10 - 24" (25.4 - 61.0cm)
	4.25" (10.8cm)
	3.75" (9.5cm)
	3.25" (8.3cm)
Weight Capacity	300lbs (136.1kg) 22-24" width 400lbs (181.4kg)

*weight based on 16 x 16"

SIZES - INCH (CM)

See Comparison Chart for sizes and product codes



- A Soft density top layer** with antibacterial properties for comfort and immersion
- B Blue Visco Gel foam** reduces heat build up critical to minimize skin integrity issues
- C Medium density Visco foam** provides improved support, positioning and immersion
- D Firm HR contour cut foam** for support, stability & durability with SPS technology

